BEABA®

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EN BATH THERMOMETER

- The BEABA bath thermometer allows you to test the temperature of bath water.
- To use the thermometer, place it in the bath water and leave it to float.
- Do not leave the bath thermometer within the reach of children.
- It is recommended that you mix the bath water well first in order to obtain the most exact measurement possible.
- Temperature indications:
 - Between 35° and 38°C: ideal temperature
 - Higher than 39°C: water too hot, risk of scalding.
- Clean with warm soapy water, rinse well and dry.

WARNING! Never leave a child alone without adult supervision.

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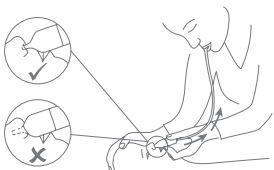
WARNING! If the liquid in the temperature column has separated, do not use the thermometer.

WARNING! This product is not a toy. Always check the condition of the product prior to use. Dispose of it as soon as you notice the first signs of wear or damage.

BABY NASAL ASPIRATOR

Instructions for use:

- Place the transparent tip at the entrance to the nostril without letting in any air.
- It is recommended that the other nostril be closed and that serum be used for greater efficiency.
- Suck up slowly.

⚠ WARNINGS

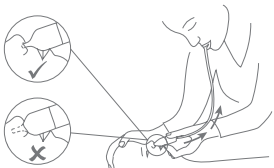
Contains small parts. For security reasons, keep out of the reach of children.

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Do not freeze the product.

FINGER GUM MASSAGER

Never leave your child unattended with the product

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To activate your guarantee, download the Béaba & Me application on App Store or Google play.